BORT Night Splint

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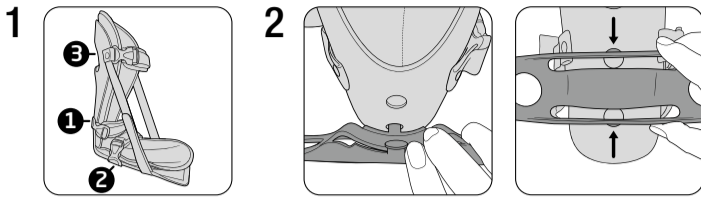
REF 100 490

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D100490I2025-10I003 ML I Rev. 01

DE BORT Night Splint

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EN BORT Night Splint

Many thanks for placing your trust in a medical device from BORT GmbH. Please read the existing instructions for use carefully. If you have any questions, please refer to your physician or the specialist retailer from whom you purchased this medical device.

Intended purpose

This medical device is a brace for positioning and fixing the foot in a defined position. It serves to immobilise and safely position the parts of the body and joints affected.

Indications

All indications for which positioning and fixing the foot is required, e.g. condition following apoplexy, infantile cerebral palsy (ICP), plantar fasciitis, Achilles tendon tendinopathy, equinus prophylaxis, postoperative immobilisation.

Contraindications

Thrombosis risk, extreme varicosis, peripheral arterial occlusive disease (PAOD), lymph drainage disorders, also unclear soft tissue swellings distal to the aid positioned, sensory and circulatory disorders of the region of the body treated, skin diseases in the part of the body treated, severe spasticity of the plantar flexors, contracture exceeding 10° of the plantar flexors.

Application risks/Important notes

- This medical device is a prescribable product. Discuss the use and duration with your treating physician. The expert staff from whom you have received the medical device will select the appropriate size and instruct you regarding its use.
- only appropriate for short walks. Ensure that you remove the tilt protection prior to use
 - remove the medical device prior to radiological examinations
 - in case of numbness, loosen the medical device and remove it if necessary
 - in case of persistent complaints, consult the physician or specialist retailer
 - use the medical device in accordance with therapeutic needs
 - only use other products simultaneously after consultation with your physician
 - do not make any changes to the product
 - do not wear it on open wounds
 - do not use in case of intolerance of one of the materials used
 - no re-use. This medical aid is intended for treating one patient only
 - whilst wearing the brace, please neither use any creams nor ointments on or around it as they can destroy the material

Fitting instructions

The product should be put on in a seated or lying position.

- 1 Open all three quick-release fasteners. Place your foot in the brace whilst ensuring that the heel is positioned at the very back of the brace rest. Close the quick-release fasteners and adapt the width of the Velcro fasteners as necessary. Use the following sequence:

1. Start with the fastener on the ankle joint.
2. Close the fastener on the foot.
3. Then, close the fastener close to the knee on the lower leg. Ensure firm and at the same time comfortable fitting of the brace.

Both Velcro fasteners on the sides of the brace can be used to increase the plantar fascia stretching: by shortening the straps; or in order to reduce stretching: by extending the straps.

- 2 The tilt protection included can be clipped on into the designated round openings above the heel in order to avoid the foot tilting away (e.g. in supine position). For this, take hold of the centre of the bar and guide both the round limit stops one after the other into the intended recesses. **Important note:** It is only possible to take short walks with the brace. **For this, ensure that you remove the tilt protection beforehand in order to minimise the risk of falling.**

The foam toe wedge can be attached between the padding and the sole using Velcro in order to increase stretching. It can be freely attached with Velcro in the anterior/posterior direction.

Information for the technician

If required, the brace rest can be formed to fit the patient's individual anatomical circumstances. For this, please remove the padding, tilt protection and toe wedge from the brace rest in advance. Use a hot air hair drier for forming with roughly:

- 160 °C
- for 3–4 min.
- at a distance of 10–15 cm.

Removal

Open the three front quick-release fasteners and remove your foot from the brace.

Material composition

Brace rest:	100% polyethylene (PE)
Padding:	60% pure ester (PUR), 40% polyester (PES)
Fasteners:	100% polypropylene (PP)
Wedge:	100% ethylene vinyl acetate (EVA)
Tilt protection:	100% polyamide (PA)

Cleaning information

Hand-washing 30 °C Do not bleach (only wash with colour or mild detergents) Do not dry in a tumble dryer Do not iron Do not clean chemically

The pad can be removed from the brace rest and hand-washed using lukewarm water. Do not use fabric conditioner. Dry in the fresh air.

Guarantee

The legal regulations of the country in which you acquired the product apply to the purchased product. Please contact your specialist retailer if you suspect a warranty claim. Please clean the product before submitting a warranty

claim. If the enclosed instructions for use have not been properly observed, the warranty may be impaired or cancelled. The warranty does not cover use of the product inappropriate for the indication, non-observance of application risks, instructions and unauthorised modifications to the product.

Useful life/Lifetime of the product

The lifetime of the medical device is determined by its natural wear and tear if treated appropriately and as recommended.

Duty of notification

If a serious worsening of the state of health occurs when using the medical device, you are obliged to notify the specialist dealer or us as the manufacturer and the Medicines & Healthcare products Regulatory Agency (MHRA). You can find our contact information in these instructions for use. You can find the contact information for the national authority responsible for your country under the following link: www.bort.com/md-eu-contact.

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Disposal

Upon the termination of use, the product must be disposed of in accordance with the corresponding local requirements.

Declaration of conformity

We confirm that this device conforms with the requirements of REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL. You can find the current declaration of conformity under the following link: www.bort.com/conformity

Status: 10.2025

 Medical device |  Single patient – multiple use

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